

Tamper resistance feature

Alaris® System



The Alaris System allows the clinician to lock the keypad on the Alaris PC unit and all attached modules. Locking the keypad minimizes the potential for inadvertent changes to the infusion program.

To use this feature, it must be enabled in your profile.


To lock the keypad (*the module must be infusing to activate this feature*):

1. Press and hold the Tamper Resistance Control on the rear of the Alaris PC unit for three seconds.
2. **When the keypad is locked**, a *PANEL LOCKED* prompt remains on the screen for three seconds.

When the keypad is locked, the clinician may:

- Silence the audio alarm
- View the Total and Pri/Sec Volume Infused
- View and test the audio alarm setting


If a clinician presses any other key or attempts to program or edit an infusion when the keypad is locked, an illegal keypress audio advisory sounds and the visual prompt *PANEL LOCKED* displays.

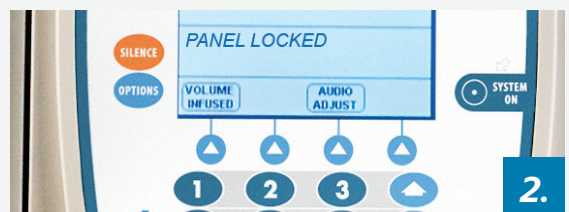

To unlock the keypad:

1. Press and hold the Tamper Resistance Control on the rear of the Alaris PC unit for three seconds.
2. **When the keypad is unlocked**, a *PANEL UNLOCKED* prompt displays on the main screen of the Alaris PC unit.

The *PANEL UNLOCKED* prompt remains on the screen for three seconds.

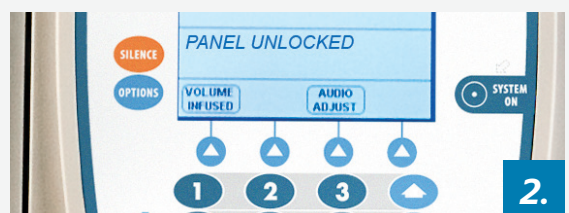

For product support, please contact Customer Advocacy at 888.812.3266 or customerfeedback@carefusion.com.

For technical support, please contact Instrument Technical Support at 888.812.3229.

For product orders, please contact Customer Order Management at 800.482.4822.

CareFusion
San Diego, CA

carefusion.com

