
UVAHealth

EPIC DOWNTIME

If EPIC has a downtime you can access the Glucommander site two ways:

- 1. Desktop ICON (G)
- KnowledgLink → Clinician Portal → View Clinician Portal →
 Connect to Other Systems → Glucommander

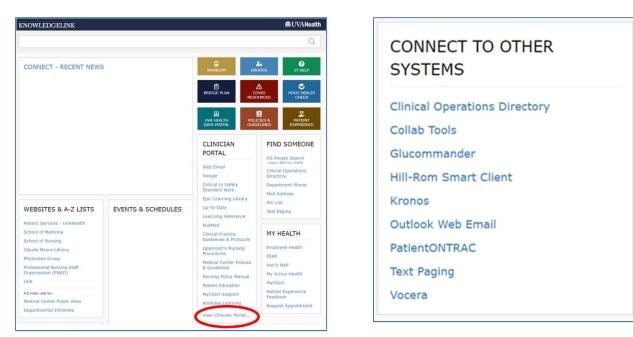


Recommended Procedure

- Obtain blood glucose 10 minutes prior to downtime
- Adjust insulin infusion per Glucommander recommendations
- Document current multiplier
- If EPIC downtime is greater than 1 hour access Glucommander site per above for next titration
- When EPIC is back up, access Glucommander via the SmartClick link on the Flowsheet or MAR and all Glucommander entries during downtime will be visible

Glucommander Electronic Glycemic Management System



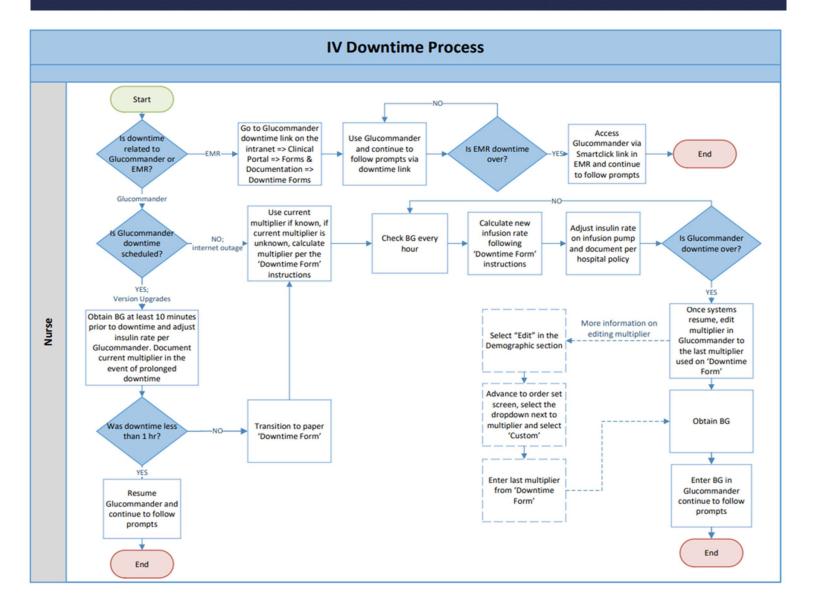

Glucommander Electronic Glycemic Management System